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Indicação:

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Characterization of Teeth

Indication:

Pigments for the intrinsic characterization of artificial acrylic teeth.

Contraindications:

There are no known adverse side effects when the product is used according to current procedures.

Warnings / Precautions:

- Follow the usual procedures for microbiological control, assembly, articulation and maintenance of the denture, as well as for individual protection.
- Use with appropriate fluid of the STG Teeth Catheterization System.
- Exclusively for dental use.
- It can form flammable air/powder mixtures.
- As in all acrylic resins, handling of the product is recommended in ventilated places, using nitrile gloves and protective goggles; however, handling is not recommendable for pregnant women, nursing mothers and people with known sensitivities to this type of product.
- Conserve by protecting from the light and heat.
- Do not dispose the wastes in sewers and watercourses or in household garbage.

Instructions for Use:

See booklet.

Characteristics / Benefits:

- Innovative characterization system of artificial teeth, developed by TPD Tomaz Gomes, through the pigmentation technique according to standards of the most common clinical cases.
- It allows recovery of naturalness in dentures.
- It is defined as a system for having scientifically developed method that allows the repetition of all proposed characterizations, such as decalcification of enamel, restorations stained by action of flavorings (tea, coffee, nicotine), enamel cracks, darkened dental colors, restorations by amalgam, etc.
- For use in photopolymerizable, self-photopolymerizable and thermopolymerizable techniques.
- Absence of porosities.
- Low water absorption.
- Low residual monomer.
- Excellent mechanical properties.
- Excellent dimensional stability.
- Unmatchable brilliance.
- Ease of drainage, compaction, ease in finishing, polishing and brilliance.

Presentations:

KIT WITH: 8 bottles of powdered resin of 3 g each; Booklet with method of application

Observation: The resins are also sold separately for replacement.

Colors:

Brown 1, Brown 2, Brown 3, Yellow, White, Blue, Gray and Colorless.

Composition:

POWDER

Polymethylmethacrylate

Benzoyl peroxide

Pigments

Shelf Life:

POWDER: 10 years from the date of manufacture.

ANVISA Registration N°:

Powder: 10216040031

Technical Responsible: Luiz Paulo Ribeiro Junior (Regional Council of Chemistry) CRQ IV - 03251745

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PULVER
Polymethylmethacrylat
Benzoylperoxid
Pigmente

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