natura
Ceram

Paste Stain & Glaze

For Color Masters

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natura Ceram

The aesthetic paste ceramic system of Natura Ceram™ gives technicians the ability to easily create aesthetic restorations with great similarity to natural teeth with ultra thin layer.

Natura Ceram™ currently works well with monolithic zirconium, lithium disilicate crowns as well as conventional porcelain layered crowns, it is easy to integrate in any laboratory work flow.

Advantage

- 1. Easy and Simple way for monolithic crowns more aesthetic.**
- 2. Easy Color Correction;** it allows you to change easily and quickly modify value & chrome.
- 3. Paste Ceramic, Alternative to powder ceramic;** an efficient way of creating highly characterized restorations that rivalize with aesthetics that normally can only be seen in Porcelain layered Crowns.
- 4. Clear & High shine Glaze with Fluorescence.**

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* Peso Neto: 5g

natura Ceram

Natura Ceram™ offers a range of 11 colors and a crystal clear glaze to help imitate the natural incisal translucency, the mamelons, pits, fissures and shiny surfaces.

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Yellow

Glaze

Pasta Stain & Glaze Temperatures from 760°C-850°C

Materials	Initial Time (°C)	Dry Time (Min)	Fire Time (°C/min)	Maximum Temperature (°C)	Retention (min)
Zirconia (monolithic)	500	05:00	50	850	01:00
Lithium Disilicate (monolithic)	500	05:00	50	820	01:00
Porcelain fused to metal	500	05:00	50	850	01:00
E-max Porcelain	500	05:00	50	760	01:00

* Illustrative images, colors may have slight variations

natura Ceram

Natura Ceram™ Paste Glaze

It is a ceramic specifically formulated to glaze dental ceramic crowns. These materials are used to glaze all kinds of dental ceramic restorations such as PFM, Zirconia, Lithium Disilicate and Porcelain.

Application:

- Dispense the amount of pasta to use.
- Wet the brush with the Glaze liquid and dry the excess with an absorbent paper.
- If necessary, add a drop or more to the paste to improve the consistency.
- Use only the Glaze liquid, do not use water because it will produce bubbles and separation in the mixture.
- Take the paste with the brush and apply evenly over the restoration.
- Cook according to the schedule of the table.

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